



GUIDE TO DEVICE CARE AND MAINTENANCE

Keep your sound processors in tip top shape using this simple guide.

TIPS FOR DAILY CARE



Keep your processor dry

- Moisture is often the main cause of sound processor problems. You can prevent this by wiping down your external processor each evening with a soft cloth and then placing it in the Dry & Store Unit.
- Set a reminder in your diary to replace the Dry Brik in your kit every two months.



Get into a cleaning routine

- **Behind-the-ear sound processors:** Cleaning your device daily will make maintenance easier. Remember to wipe down your processor, coil cable, magnet coil and earhook with a soft dry cloth every day. You do not have to disconnect the earhook or coil cable.

TIP: after wiping down all the components, make sure the coil cable is still fully inserted into the socket on the coil. A loose cable can be the cause of a range of issues, like intermittent sound, or even no sound at all.

- **Off-the-ear sound processors:** If you have an off-the-ear sound processor, simply wipe down the sound processor with a soft cloth.



Check your microphone cover

- Distorted or unclear sound might be caused by something as simple as a dirty microphone protector, so check for signs of dirt or grime every night.
- To maintain clear sound we recommend replacing your microphone cover every three months (or as needed). It's very important to change microphone protectors regularly because they keep particles out of your processor unit. Over time, they block up, so when you change them, you will find the sound is much louder and clearer.

TIPS FOR BATTERY USAGE



Check for signs of corrosion daily

- In warmer or more humid climates, corrosion can be a common cause of problems with sound processors. So, remember to clean the battery connections and battery daily.



Maintenance will depend on the type of battery you use

- There are two types of batteries: disposable zinc air batteries and rechargeable batteries which can last for at least 365 charge cycles.

TIP: For rechargeable batteries, remember that a completely empty rechargeable battery will take about four hours to recharge. Make sure you have spare batteries available and ready.



Rechargeable batteries will require a bit of extra care

- Check that you have inserted only rechargeable batteries into your charger.
- If the battery charger is splashed with liquid, carefully shake out the liquid and dry the battery charger thoroughly with a dry soft cloth. Leave it in a dry area for 24 hours. Don't use the charger until it's completely dry.
- Remember to give your charger a good clean regularly. Unplug your charger, turn it upside down and tap it to remove dirt. Wipe the charger sockets with a soft, dry cloth.
- When not in use, turn the battery charger over so it is facing down. For the Nucleus® 7 Sound Processor USB charger, put the cover on the connector when not in use. This will help prevent dust/dirt accumulating in the battery charger sockets.

TIP: To ensure you do not wear out one socket, cycle your batteries in different sockets when you recharge.

For more information contact your local customer service team or get connected with Cochlear Family to receive updates on device usage.

Please seek advice from your health professional about treatments for hearing loss. Outcomes may vary, and your health professional will advise you about the factors which could affect your outcome. Always read the instructions for use and refer to the relevant user guide for more information. Not all products are available in all countries. Please contact your local Cochlear representative for product information. Consult your health professional to determine if you are a candidate for Cochlear technology.

© Cochlear Limited 2020. Cochlear, Hear now. And always, the elliptical logo, and marks bearing an ® or ™ symbol, are either trademarks or registered trademarks of Cochlear Limited or Cochlear Bone Anchored Solutions AB (unless otherwise noted).

www.cochlear.com

Cochlear Ltd (ABN 96 002 618 073) 1 University Avenue, Macquarie University, NSW 2109, Australia Tel: +61 2 9428 6555

Cochlear NZ Limited Level 4, Takapuna Towers, 19-21 Como Street, Takapuna, Auckland 0622 New Zealand Tel: +64 9 914 1983

株式会社日本コクレア (Nihon Cochlear Co Ltd) 〒113-0033 東京都文京区本郷2-3-7 お茶の水元町ビル Ochanomizu-Motomachi Bldg, 2-3-7 Hongo, Bunkyo-ku, Tokyo 113-0033 Japan Tel: +81 3 3817 0241

Cochlear (HK) Ltd Room 1404-1406, 14/F, Leighton Centre, 77 Leighton Road, Causeway Bay, Hong Kong Tel: +852 2530 5773

Cochlear Medical Device (Beijing) Co Ltd Unit 2608-2617, Gemdale Tower B, 91 Jianguo Road, Chaoyang District, Beijing, 100022, P.R. China Tel: +86 10 5909 7800

Cochlear Limited (Singapore Branch) 238A Thomson Road, #25-06, Novena Square Office Tower A, Singapore 307684 Tel: +65 6553 3814

Cochlear Korea Ltd 1st floor, Cheongwon Building 33, Teheran-ro 8 gil, Gangnam-gu, Seoul, Korea Tel: +82 2 533 4450

Cochlear Medical Device Company India Private Ltd G Floor, Platina Building, Plot No C-59, G-Block, Bandra Kurla Complex, Bandra (E), Mumbai - 400 051, India Tel: +91 22 6112 1111

D1702538 MAR20

